


IPROPULSE®

Next Generation Ear Care **NG**

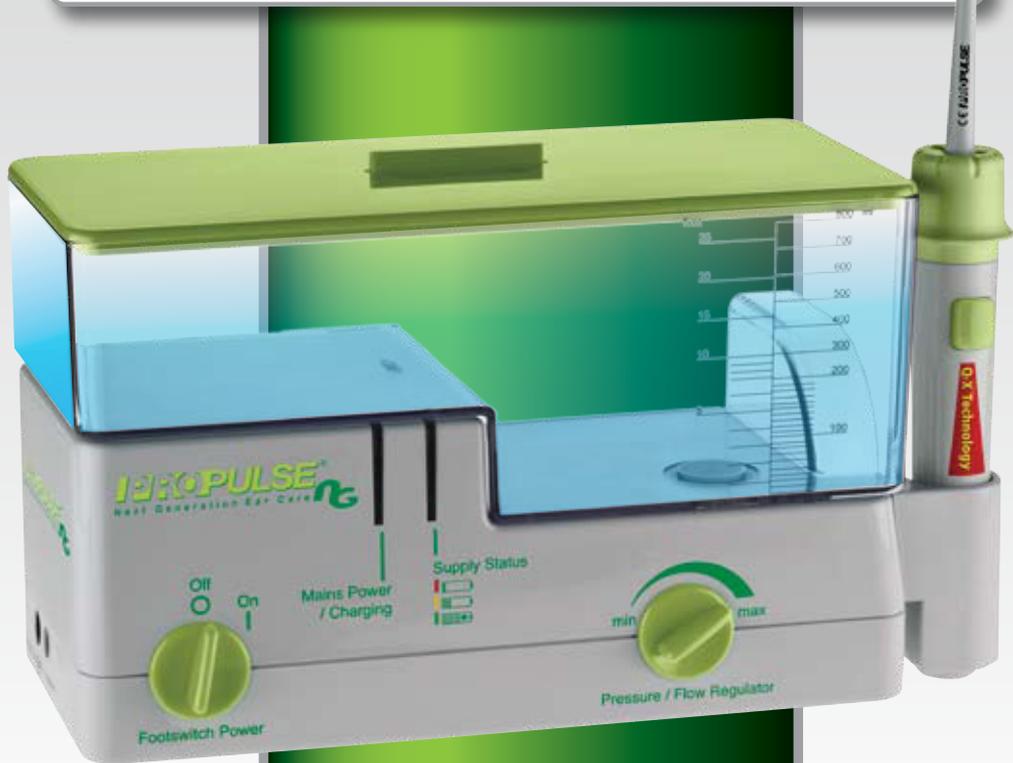

Uses the patented
CONTRAPULSE®
safety system

QrX™ Technology



emech
+ + +
m e d i c a l

New Patented Quick-Release Handle & QrX™ Single-Use Tips

The Propulse® Ng Electronic Ear Irrigator is now supplied with the all-new, patented Quick-Release Handle & QrX™ Single-Use tips. The QrX™ system locks the tip into place during use making this the most operator-friendly Propulse® Ng to-date!

To help prevent cross infection between patients, clinicians must use a new tip for each treatment.*

New QrX™ Tip



Supplied in packs containing 100 Single-Use, individually wrapped, non-sterile tips.

Propulse® Ng QrX™ Replacement Tips (#100) KIT4002

*National Guidelines for Ear Care — PECC.
www.earcarecentre.com/protocols.htm



Electronic ear irrigation system

The Propulse® Ng sets new standards in ear irrigation. Its patented and unique features create an irrigation system that is safe, convenient and patient-friendly.

- *Quick-Release Handle*
- *Single-use QrX™ Tips*
- *Contrapulse® water delivery system*
- *Trusted Propulse® Quality*

Why Propulse® Ng ear irrigator?

*"The metal syringe is obsolescent for use in the ear canal. The syringe design is inherently dangerous"**

*"Electronic ear irrigators such as the Propulse® allow irrigation of the ear canal rather than wax removal under pressure"**

*To read the report in full, log onto www.entnursing.com/earcare.htm

The Propulse® Ng Irrigation System is both portable and rechargeable making it ideal for domiciliary visits or in areas with a questionable electricity supply. Its variable pressure delivery system provides a pulsed water jet, which is easily controlled.

The Propulse® Ng Electronic Ear Irrigator is now supplied with the all new, patented Quick-Release Handle & QrX™ Single-Use tips. The QrX™ system locks the tip into place during use making this the most operator-friendly Propulse® to-date!

The operation of the Propulse® Ng is controlled by the convenient footswitch, allowing the machine to be controlled hands free. The flow of water can be stopped by fully depressing the water jet stop button – water flow resumes when it is released.

Propulse® Ng QrX™ Tips and Propulse® Cleaning Tablets ensure that recommended levels of hygiene are maintained and that the risk of cross infection between patients is minimised.

Propulse® Ng features



- Propulse® Ng ear irrigator is the safe alternative to the traditional manual ear syringing procedure
- Propulse® Ng ear irrigator helps facilitate patient friendly, fast and efficient cerumen, keratin and foreign body removal from the external auditory meatus
- Quieter operation helps improve patient experience during the ear care procedure
- Propulse® Ng comes with a 12 month parts & labour warranty
- Propulse® Ng ear irrigator meets all the EC safety standards

QrX™ Technology
is protected by a number
of patents pending.

Intended use

The Propulse® Ng Electronic Ear Irrigation System is a Class IIa medical device. Its intended use is for the removal of cerumen, keratin, or any other (non-hygroscopic) foreign body from the human ear by means of irrigation with water. The Propulse® Ng Electronic Ear Irrigation System is CE Marked and complies with

- Medical Devices Directive – 93/42/EEC
- Electrical Safety – EN60601-1-2
- Electro Magnetic Compatibility

Each Propulse® Ng ear irrigation system comprises:



- Propulse® Ng Unit
- Integrated QrX™ Handle
- Propulse® Footswitch
- 10 - Propulse® QrX™ Single-Use Tips
- 10 - Cleaning Tablets (via registration card)
- Power Supply
- Internal Rechargeable Power Pack
- User Manual

Propulse® Chlor Clean cleaning tablets

Cleaning tablets for your Propulse® Ear Irrigator – the importance of using the correct strength of cleaning solution cannot be overstated. A solution that is too strong will in time damage the Propulse®, and one that is too weak will fail to provide the correct level of cleaning and decontamination. Propulse® cleaning tablets have been formulated to provide the exact strength of cleaning solution that is required to clean your Propulse® effectively and safely.

Please note: Damage caused to your Propulse® Ng Ear Irrigator as a result of not using the recommended cleaning agent will invalidate your warranty.



(#200) CL0001